

KRUUSE Vet-Flex & Fun-Flex Bandages

Strong, comfortable and easy-to-tear cohesive bandages

Features and benefits

Reliable and stable fixation

The high weight of the material, 35 g/m², makes the bandage strong and firm, yet comfortable. It also gives excellent compression when you need it most.

Easy to tear

No scissors needed! Simply pull in opposite directions to tear in a straight line.

Skin-friendly

KRUUSE Vet-Flex is coated with latex, which makes the cohesiveness stick to itself, but not to other surfaces like skin or hair. The bandage is also porous allowing the fabric to breathe.

Very flexible

The KRUUSE Vet-Flex can be stretched to almost double the unrolled length! You can bandage all kind of injuries on the animal's body and control the compression.

Excellent elasticity

The bandage has spandex threads that help minimize shrinkage to give a firm compression and fixation. With proper application, the bandage will not constrict.

Easy to remove

The features of the KRUUSE Vet-Flex cohesive bandage make it very easy to remove, even without scissors.

Hygienic

The bandage is water-resistant, which helps it stay in place. Each roll is individually wrapped in an easy-to-tear pouch for your convenience.

Good compliance from the patient

The bandage is very light, does not constrict and is not sticky to touch.



Easy to tear



Latex threads



Spandex threads

Recommended use for KRUUSE Fun-Flex

- Controlled support and compression of joints, sprains, distortions
- Fixating dressings

Instructions for use

Application

Fixate the end of the bandage and apply evenly over the injured body part. Stretch the bandage according to the level of compression needed. When finished applying, tear the bandage by holding straight and pulling apart in opposite directions. Do not twist! Gently squeeze the layers together to enhance the cohesiveness. Use KRUUSE Fun-Flex as secondary or tertiary bandage on top of primary or secondary dressings.

Removal

Unroll bandage or cut gently with bandage scissors.

Do not use

If patient or user is allergic to latex.

For veterinary use only

KRUUSE Fun-Flex Bandages

Cat No. Description

- 164002 KRUUSE Fun-Flex Pet bandage, assorted, 5 cm, 10 rolls
- 164004 KRUUSE Fun-Flex Pet bandage, assorted, 7,5 cm, 10 rolls
- 164006 KRUUSE Fun-Flex Pet bandage, assorted, 10 cm, 10 rolls
- 164008 KRUUSE Fun-Flex Summer, 5 cm x 4.5 m, assorted, 10/pk
- 164009 KRUUSE Fun-Flex Summer, 7.5 cm x 4.5 m, assorted, 10/pk
- 164011 KRUUSE Fun-Flex Summer, 10 cm x 4.5 m, assorted, 10/pk



KRUUSE Vet-Flex Bandages

Cat No. Description

- 160739 KRUUSE Vet-Flex Pet bandage, assorted bright, 10 cm, 10 rolls
- 160740 KRUUSE Vet-Flex Pet bandage, assorted bright, 7,5 cm, 10 rolls
- 160741 KRUUSE Vet-Flex Pet bandage, assorted bright, 5 cm, 10 rolls
- 160742 KRUUSE Vet-Flex 15 cm x 15 m, red, 4/pk
- 160743 KRUUSE Vet-Flex 15 cm x 15 m, blue, 4/pk



For veterinary use only